

TURBOGAL[®] spray

Spray-oil for lubrication of all types of handpieces

COMPOSITION

Liquid paraffin, isobutane

CHARACTERISTICS

Turbogal spray-oil is very effective for daily cleaning and lubrication of all types of handpieces used in dentistry.

The oil is odorless and tasteless.

METHOD OF USED

For professional use.

The handpieces used in dentistry be lubricated every day before the first and after the last use, and also before each sterilization. Hold the Turbogal spray bottle upright and spray the inside of the instrument for one second. Remove excess oil from the outer surface of the instrument.

NOTE

Keep away from heat, sparks, open flame, or other sources of ignition and hot surfaces. The container is under pressure, do not pierce or burn it even when it is used. No smoking during use. When spraying, keep the bottle away from the eyes and do not inhale.

STORAGE

Store in the original package at a temperature up to 25°C, protected from heat sources, sunlight and flammable materials.

SHELF LIFE

3 years.

Do not use this product after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

PACKAGING

Spray-bottle with 250ml of oil.



11000 Beograd, Višnjički venac 73D