

BAZOGAL® G and D
Upper and Lower Base Plates**COMPOSITION**

1 g of mass contains:

Shellac 466 mg

CHARACTERISTICS

Shellac base plate for the upper jaw is in the shape of trapezium, while the one for the lower jaw is in the form of a horseshoe. Bazogal G and D plates are manufactured using 1.5 – 1.65 mm thick thermoplastic mass, and are dimensionally stable. The material is firm, in thin layers. The colour of the plates is light pink. At room temperature, the plates break when bending is attempted. When warmed up, they become plastic and easy to mould. They adapt easily on working model, do not soften at mouth temperature and do not deform during preparation of dentures. After cooling, they retain the moulded shape.

INDICATIONS

The make of bite registration matrix base.

Bite registration matrix comprises a base and a bite rim, and is used to:

- determine the bite relationship of the jaws as well as shape and position of the artificial dental arches;
- transfer models into articulator;
- position teeth.

They can also be used as individual trays when taking functional impression.

CONTRAINDICATIONS

Not known.

PRECAUTIONS

No special precautions.

DOSAGE AND METHOD OF USE

All undercut portions of the final working model should be filled with the appropriate material. Isolate the prepared model by using talc or by immersing it briefly into water. Heat the shellac base plate in the flames to make it plastic, and work it with damp fingers until close contact with the surface of the model is achieved.

Cut the excess of base plate on the edges.

NOTE

When heating the flask containing denture with wax, pay attention not to leave it in boiling water for longer than 3 minutes. If left longer than that, the wax melts and Bazogal softens in excess, thus making its removal from the teeth and plaster in the flask difficult (it sticks to them).

Being inflexible, shellac base plate is desirably supported with wire at the pharyngeal ridge and on the lingual side.

Only for professional use.

ADVERSE EFFECTS

Not known.

STORAGE

Store in the original package 10°C – 25°C.

SHELF LIFE

5 years.

Do not use this medical device after the date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

PACKAGING

Specially

Box with 12 base plates of Bazogal G

Box with 12 base plates of Bazogal D

Decision No.: 515-02-00154-20-003

Date of the Instruction approval: March 04, 2020.

Authorised Representative for Europe:

EC	REP
----	-----

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia



Tel (+381 11) 40 80 108

E-mail: dental@profarm.rs

www.profarm.rs