



## **DENTAL PAPER ROLLS**

**Not Sterile**

### **COMPOSITION**

Cellulose 100%

### **CHARACTERISTICS**

Dental paper rolls are paper rolls in the form of white cylinders, used in dentistry to absorb saliva in various dental procedures. Paper rolls are not sterile, but are microbiologically approved.

### **USAGE**

Dental paper rolls are used during dental procedures to separate tooth, or group of teeth from the oral tissue (tongue, face) in order to prevent wetting of the work field, to improve access and visibility of the working field.

After the end of use, prior to removal from the oral cavity, additionally soak the paper roll with water in order to avoid its adhering to the mucous membrane of the mouth.

### **NOTE**

Paper rolls must not be placed directly onto an open wound.

### **STORAGE**

Store in the original package at temperatures between 10°C and 25°C in a dry and clean place, protected from direct sunlight.

### **SHELF LIFE**

5 years.

Do not use this medicine after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

## PACKAGING

Bag with 400 paper rolls

Decision No.: 515-02-00255-20-004

Date of the Instruction approval: March 06, 2020.

Authorised representative for Europe:

EC	REP
----	-----

**EMERGO EUROPE**  
**Prinsessegracht 20**  
**2514 AP The Hague**  
**The Netherlands**

Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia



Tel (+381 11) 40 80 108

E-mail: dental@profarm.rs

[www.profarm.rs](http://www.profarm.rs)