



**KUPROVENT® N**  
**Thermoplastic Impression Material**

**COMPOSITION**

1 g of mass contains:

|                       |           |
|-----------------------|-----------|
| Natural Resin (Copal) | 196.94 mg |
| Resenol 120           | 226.39 mg |

**CHARACTERISTICS**

Kuprovent N is manufactured in the form of metallic brown sticks. The material belongs to a group of thermoplastic composite impression materials, type I, that melt at temperatures of 55°C to 60°C. Owing to the presence of metal particles, the plastic phase is prolonged at temperatures of 45°C to 50°C, providing easy and comfortable manipulation. Impressions obtained with Kuprovent N are homogenous, with smooth surfaces.

**INDICATIONS**

Kuprovent N is used for:

- formation of a vent margin in functional impressions of edentulous jaws;
- correction of individual trays.

**CONTRAINDICATIONS**

Not known.

**PRECAUTIONS**

No special precautions.

**DOSAGE AND METHOD OF USE**

Heat Kuprovent N sticks with a water bath at a temperature of approximately 60°C. Take the impression using a warm and softened Kuprovent N as per stated indication or plan of therapy.

Impressions taken with Kuprovent N thermoplastic mass should be stored at room temperature until definite impression is made.

**NOTE**

Do not overheat Kuprovent N on an open flame. This process leads uneven heating and disintegration of the material which loses its favorable properties required for a good impression.

Close the box immediately after use.

Only for professional use.

## **ADVERSE EFFECTS**

Not known.

## **STORAGE**

Store in the original package at temperatures between 10°C and 25°C.

## **SHELF LIFE**

5 years.

Do not use this medical device after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

## **PACKAGING**

Box with 18 sticks

Decision No.: 515-02-04387-19-003

Date of the Instruction approval: January 20, 2020.

Authorised representative for Europe:

|    |     |
|----|-----|
| EC | REP |
|----|-----|

**EMERGO EUROPE**  
**Prinsessegracht 20**  
**2514 AP The Hague**  
**The Netherlands**

Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia



Tel (+381 11) 40 80 108

E-mail: dental@profarm.rs

www.profarm.rs

P029