

**RETRARGIN<sup>®</sup> Solution****Gingival Retraction Solution****COMPOSITIONS**

1 ml of solution contains:

Aluminium Chloride Hexahydrate	250.00 mg
8-Hydroxyquinoline Sulphate	1.00 mg

**CHARACTERISTICS**

The active ingredient of Retrargin solution is a hypertonic solution of aluminium chloride which has astringent effect. A cord soaked in the solution and packed around the prepared teeth causes rapid opening of gingival sulcus and retraction of gingiva.

**INDICATIONS**

Retrargin solution is used for wetting the cord for rapid and temporary gingival retraction of gingival sulcus and obtaining a dry and clear area:

- before taking the secondary, correction impression using elastomers;
- in the preparation of subgingival cavity margins.

**CONTRAINDICATIONS**

None known.

**PRECAUTIONS**

Make sure to avoid skin contact. Wear protective gloves.

**DOSAGE AND METHOD OF USE**

Soak a suitable length of retraction cord in the solution and pack the cord carefully into gingival sulcus around the prepared teeth, starting from the oral surface, over both approximal surfaces, ending finally at vestibular surface. The soaked cord must not be allowed to remain in the sulcus for long. If taking impressions of multiple prepared teeth, first pack the cord into the sulcus and then soak the loose end of the cord in the solution, using the tip of the tweezers.

**NOTE**

Close the bottle immediately after use.  
Only for professional use.

**ADVERSE EFFECTS**

None known.

## STORAGE

Store at the temperature of 15°C to 30°C, in the original package.

## SHELF LIFE

3 years.

Do not use this medical device after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

## PACKAGING

A bottle of 20 ml of solution.

Decision No.: 515-02-04387-19-003

Date of the Instruction approval: January 20, 2020.

Authorised representative for Europe:

EC	REP
----	-----

**EMERGO EUROPE**  
**Prinsessegracht 20**  
**2514 AP The Hag**  
**The Netherlands**

Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia



Tel (+381 11) 40 80 108

E-mail: dental@profarm.rs

www.profarm.rs

