



SALIGAL®
Dental Suction Tips for Saliva Aspiration

COMPOSITION

Tube: polyvinyl chloride	70.00%
phthalates	25.40%
Cap: polyethylene	

CHARACTERISTICS

Dental suction tips, Saligal, are manufactured in soft, transparently PVC material, reinforced with a delicate wire. They are flexible, non-toxic and latex-free.

DIRECTIONS FOR USE

Gently curve Saligal tip on its end covered with a cap. Connect the other, free end of a tip to the suction tubing of the aspirator. Place the capped part of a tip in patient's mouth.

NOTE

After every single use of Saligal make sure that you dispose of a suction tip.
Only for professional use.

STORAGE CONDITIONS

Store in the original packaging 15°C – 30°C.

SHELF LIFE

5 years

Do not use this medicinal device after the expiry date indicated on the packaging.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

HOW SUPPLIED

A bag containing 100 pieces of suction tips.

Available as colorless, in green, red and blue colour.

Decision No.: 515-02-00255-20-004

Date of the Instruction approval: March 06, 2020.

Authorised representative for Europe:



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia
Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia

Tel (+381 11) 40 80 108
e-mail: dental@profarm.rs
www.profarm.rs

