

**VANTAL<sup>®</sup> Paste****Soft Deposits Removal  
and Teeth Polishing Paste****COMPOSITION**

1 g of paste contains:

Calcium Carbonate	163 mg
Pumice Powder	444 mg
Flavour	2 mg

**CHARACTERISTICS**

Vantal paste is a pink paste of soft consistency, containing uniformly distributed abrasive particles.

The paste removes effectively the soft deposits from the teeth (dental pellicle, dental plaque, materia alba, tobacco stains and other deposits).

It enables teeth polishing without causing damage to the enamel.

Cleansed and polished teeth remain smooth for certain period, thus reducing dental plaque accumulation.

Vantal paste is of refreshing aroma.

**INDICATIONS**

Removal of soft deposits:

- in the prevention of gingivitis and periodontal disease;
- in the treatment of periodontal disease;
- following the treatment of periodontal disease;
- in the prevention of dental caries;
- prior to application of composite materials.

**CONTRAINDICATIONS**

None known.

**PRECAUTIONS**

No special precautions required.

**DOSAGE AND METHOD OF USE**

Dental calculus need to be removed thoroughly prior to Vantal paste application.

Squeeze the appropriate amount of the paste out of the tube and apply to teeth surfaces by means of a suitable instrument.

Use rotary instruments (brushes and rubber cups) running at a low rpm to cleanse and polish the teeth, making sure Vantal paste is constantly applied.

Caution should be taken to avoid any damage to gingival tissue.

## NOTE

Close the tube immediately after use.  
Only for professional use.

## ADVERSE EFFECTS

None known.

## STORAGE

Store at the temperature 15°C to 30°C, in the original package.

## SHELF LIFE

3 years.

Do not use this medical device after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

## PACKAGING

A tube of 40 g of paste

Decision No.: 515-02-04387-19-003

Date of the Instruction approval: January 20, 2020.

Authorised representative for Europe:



**EMERGO EUROPE**

**Prinsessegracht 20**

**2514 AP The Hague**

**The Netherlands**



Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia

Tel (+381 11) 40 80 108

e-mail: [dental@profarm.rs](mailto:dental@profarm.rs)

[www.profarm.rs](http://www.profarm.rs)

