



VOMOGAL[®] S
Dental Modelling Wax

COMPOSITION

Paraffin Type N 5803
Paraffin, White, Soft
White Wax
Carnauba Wax
White Ceresin (melting point 80-90°C)
Balsam Resin
Ponceau Colour 4R

CHARACTERISTICS

Vomogal S is of a complex chemical composition, being a blend of natural and synthetic waxes. All raw materials contained in the product are of standard quality. Vomogal S is manufactured in the form of pink, non-scaling, non-sticky plates, which do not stain plaster. Vomogal S is of neutral taste and odour. It enables simple and easy modelling at different room temperatures.

The plastic deformation (flow) of Vomogal S is:

at 23°C	NMT 1.0 %
at 37°C	5 % – 90 %

INDICATIONS

Vomogal S has a wide range of applications in dentistry and dental laboratory technology. It is used for denture plates modelling, fabrication of occlusion rims, bite registrations and other applications in dental laboratories and dental offices.

CONTRAINDICATIONS

None known.

PRECAUTIONS

No special precautions required.

DOSAGE AND METHOD OF USE

Soften the required amount of Vomogal S plate over a flame and use as indicated.

NOTE

Close the box immediately after use.
Only for professional use.

ADVERSE EFFECTS

None known.

STORAGE

Store below 25°C, protected from light.

SHELF LIFE

3 years.

Do not use this medical device after the expiry date indicated on the package.

Any used or unused product which is beyond the expiry date should be handled in accordance with the applicable legal regulations.

PACKAGING

A box of 400 g of Vomogal S plates.

Decision No.: 515-02-04387-19-003

Date of the Instruction approval: January 20, 2020.

Authorised Representative for Europe:

EC	REP
----	-----

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Entry holder: Profarm d.o.o., 11000 Belgrade, Nemanjina 36, Serbia

Manufacturing site: Profarm d.o.o., 11000 Belgrade, Višnjički venac 73D, Serbia

Tel (+381 11) 40 80 108

E-mail: dental@profarm.rs

www.profarm.rs

